

PlasmaPrep 12™



PlasmaPrep 12™

Fast serum and plasma separation

- PPP in 2 minutes
- PRP in 30 seconds
- 12 place rotor centrifuge with built in safety features
- Whisper quiet with excellent build quality

Consistent, high quality preparation at your fingertips.

<p>Easy to use</p>	<ul style="list-style-type: none"> • 12 place rotor • Four preprogrammed spin times: <ul style="list-style-type: none"> - 30 seconds for Platelet-Rich-Plasma - 2 minutes for Platelet-Poor-Plasma - 3 minutes for intermediate spin - 5 minutes for hard spins • Accepts 13 Ø x 75 (h) mm through 16 Ø x 100 (h) mm tubes
<p>Fast and accurate</p>	<ul style="list-style-type: none"> • Rapid processing. Tubes can be processed as they arrive, no need to wait for batches • Whisper quiet • Heavy duty, Swiss made motor for industry standard reliability • Fast separation without high speed risk for COAG or STAT • 2 year warranty
<p>Safety Features</p>	<ul style="list-style-type: none"> • Locking lid with gasket seal • Operates only when the lid is locked into position • Lid remains locked at all times during rotation • Removable rotor for easy cleaning • Disposable tube holders keep the centrifuge bowl clean • Automatic out-of-balance shut down
<p>Dimensions</p>	<ul style="list-style-type: none"> • Dimensions: 254(w) x 301(d) x 254 mm(h) / 10"(w) x 15"(d) x 10"(h) • Weight: 7.25 kg / 17 lb




1

Insert tube holders into rotor.



2

Invert sample 3 times before placing into tube holder. Ensure counterbalance sample is also used.

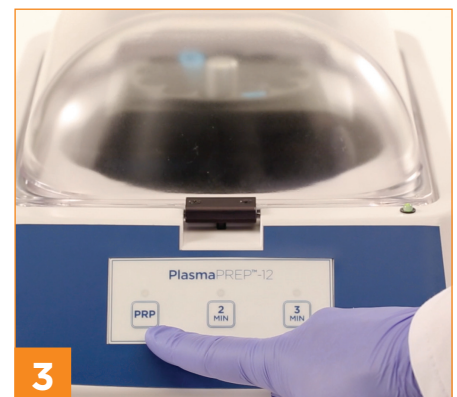

3

Close lid and select required spin cycle.

Distributed by

Manufacturer
 Separation Technology
 570 Monroe Road, Suite 1008
 Sanford, FL 32771, USA
custserv@separationtechnology.com
 +1 (407)-788-8791
ekfdiagnostics.com