



## True® 20 & True® 20 Plus

Truly accurate, truly economical,  
truly exceptional

- ◆ Easy-to-read results
- ◆ Convenient room temperature storage
- ◆ Rapid results
- ◆ Urine and serum samples

# True<sup>®</sup> 20 & True<sup>®</sup> 20 Plus

Everything you need in a convenient pouch (including dropper).

## Exceptional readability

Quick clearing background ensures easy-to-read results even with low-level specimens

## Fast

Positive results in as little as 10 seconds\*

## Sensitive

Detects 20 mIU/mL hCG or greater in both urine and serum specimens

## Convenient



Room temperature storage

## Reliable, specific and accurate

Specific mono and polyclonal antibodies ensure early pregnancy detection while preventing cross reactivity with other hormones present in patient samples

## Test procedure

- Remove True<sup>®</sup> 20 / True<sup>®</sup> 20 Plus device and disposable dropper from the foil pouch
- Label the test device with patient or control identification
- Dispense two drops of the specimen into the round sample well
- Read result as positive or negative

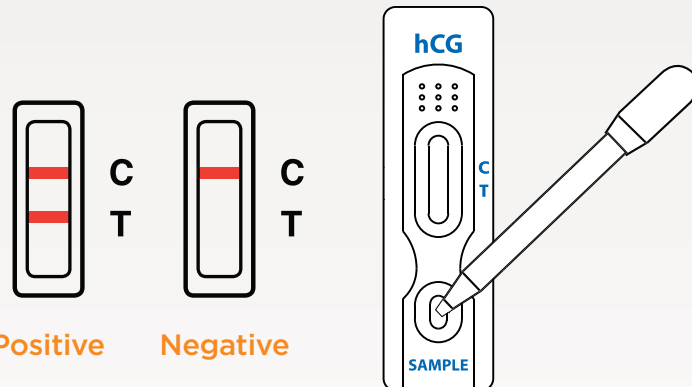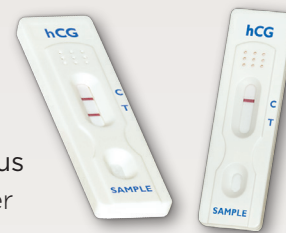

Reference No.

1430-050	True <sup>®</sup> 20 hCG (Urine)	(50 tests)
1440-050	True <sup>®</sup> 20 Plus hCG (Serum/Urine)	(50 tests)



Distributed by



for Urine specimens

CODE\*: 81025



Stanbio

1261 North Main Street  
Boerne, Texas 78006, USA

Email: sales2@ekfdiagnostics.com

+1 830 249 0772

(USA toll free) 1 800 531 5535

www.ekfusa.com

ISO 13485:  
2003 Certified

**EKF** | Diagnostics  
for life

\* The test is positive if two colored lines appear. One colored line will appear at the Specimen Zone and one at the Control Zone.

\*\* Exemplar code provided. Refer to regional CMS Fee Schedule for relevant codes and reimbursement rates.