



True® 20 & True® 20 Plus

Truly accurate, truly economical,
truly exceptional

- ◆ Easy-to-read results
- ◆ Convenient room temperature storage
- ◆ Rapid results
- ◆ Urine and serum samples



for Urine specimens



Diagnostics
for life

True[®] 20 & True[®] 20 Plus

Everything you need in a convenient pouch (including dropper).

Exceptional readability

Quick clearing background ensures easy-to-read results even with low-level specimens

Fast

Positive results in as little as 10 seconds*

Sensitive

Detects 20 mIU/mL hCG or greater in both urine and serum specimens

Convenient



Room temperature storage

Reliable, specific and accurate

Specific mono and polyclonal antibodies ensure early pregnancy detection while preventing cross reactivity with other hormones present in patient samples

Test procedure

- Remove True[®] 20 / True[®] 20 Plus device and disposable dropper from the foil pouch
- Label the test device with patient or control identification
- Dispense two drops of the specimen into the round sample well
- Read result as positive or negative

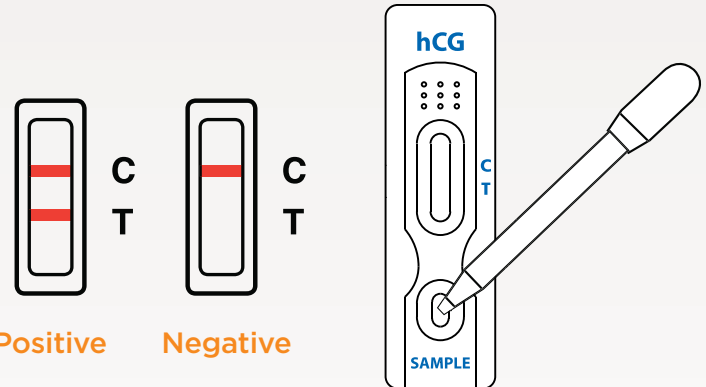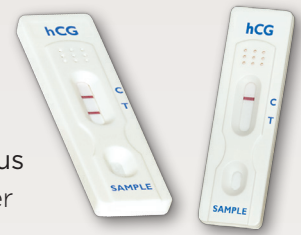

Reference No.

1430-050	True [®] 20 hCG (Urine)	(50 tests)
1440-050	True [®] 20 Plus hCG (Serum/Urine)	(50 tests)



Distributed by



for Urine specimens

CODE*: 81025



Stanbio

1261 North Main Street
Boerne, Texas 78006, USA

Email: sales2@ekfdiagnostics.com

+1 830 249 0772

(USA toll free) 1 800 531 5535

www.ekfusa.com

ISO 13485:
2003 Certified



Diagnostics
for life

* The test is positive if two colored lines appear. One colored line will appear at the Specimen Zone and one at the Control Zone.

** Exemplar code provided. Refer to regional CMS Fee Schedule for relevant codes and reimbursement rates.