



## STAT-Site® M Hgb

A truly portable hemoglobin testing device

- ◆ Optimum performance and reliability
- ◆ Hemoglobin result within 50 - 120 seconds
- ◆ Pocket sized device
- ◆ One battery performs approximately 1,000 tests

## Fast and precise hemoglobin results are provided in seconds using a single drop of capillary blood

<p>Easy-to-use</p>	<ul style="list-style-type: none"> <li>• User friendly features minimize training time</li> <li>• Step by step instructions on screen</li> <li>• 1 drop (approximately 12 <math>\mu</math>L) of whole blood</li> <li>• Virtually maintenance free</li> <li>• 'Last result' feature allows recall of the previous hemoglobin measurement</li> </ul>
<p>Accurate and reliable</p>	<ul style="list-style-type: none"> <li>• Reflectance photometry azide-methemoglobin method</li> <li>• Total precision CV: 2.9 % - 4.2 %</li> <li>• Within run precision CV: 3.5 % - 4.9 %</li> <li>• Linearity: 5.6 - 20.6 g/dL (3.48 - 12.78 mmol/L)</li> <li>• Results within 50 - 120 seconds</li> </ul>
<p>Practical and portable</p>	<ul style="list-style-type: none"> <li>• Pocket-size: 17 mm (h) x 55 mm (w) x 100 mm (d)</li> <li>• Lightweight: 57 g (0.2 oz)</li> <li>• 1,000 tests using just 1 x 3V Li (CR 2032) type battery</li> <li>• Operating range: 16 ° - 35 °C (61 ° - 95 °F)</li> <li>• Humidity: 20 % - 80 % (non-condensing)</li> <li>• Automatic calibration (uses CODE key)</li> </ul>



## Easy as one, two, three



1 Insert the test card



2 Apply the sample



3 Read the result

Distributed by

CE

CLIA'SB  
WAIVED

Manufacturer

Stanbio  
1261 North Main Street  
Boerne, Texas 78006 USA  
Tel. (Toll Free USA Only) 800 531 5535  
Tel. +1 830 249 0772  
Email: sales2@ekfdiagnostics.com

[www.ekfdiagnostics.com](http://www.ekfdiagnostics.com)

**EKF** | Diagnostics  
for life

Revision EN EU 3.0-03-2016