



Lactate Scout Vet

Lactate analyzer for veterinary care

Scan here to view



product webpage

Result in 10 seconds

Stores up to 500 results

Only a single drop of blood required

Simple to use

Ergonomic design, small, light and robust

EKF
Diagnostics

Lactate Scout Vet

Easy-to-Use

- Insert/remove test strip to turn on/off
- Select animal species: Feline, Canine, Porcine, Ovine, Bovine, Equine
- Only 0.2 µl of capillary or whole blood required
- Pre-calibrated test strips
- Device calibrated by simple coding
- Lactate concentration, date/time, measuring mode, temperature and memory ID on a single display
- Automatic self test

Fast and accurate

- Enzymatic amperometric detection method
- Results within 10 seconds
- Measuring range: 0.5 – 25 mmol/L
- Lactate Scout Check Solutions available for function control

Performance measurement

- Prognosis and diagnosis of a variety of animal health conditions (such as shock, severe sepsis, low cardiac output, liver failure, poisoning, and drug therapy)
- Assess the effects of preslaughter handling and transport stress
- Performance diagnostics for high performance and race horses

Practical and reliable

- Stores up to 500 results
- 1,000 tests using just 2 x CR2450 batteries
- Pocket-size: 91 mm (h) x 46 mm (w) x 21 mm (d)
- Lightweight: 60 g
- Operating range: 10 – 45°C and max. 85% humidity

Catalog no.

Name

7023-0512

Lactate Scout Vet

7023-3422

Lactate Scout Test Strips (new 25 pack)

7023-6300

Lactate Scout Check Solution Low (1.8 - 2.2 mmol/L) - 1 x 2.5mL bottle

7023-6301

Lactate Scout Check Solution Mid (4.5 - 5.6 mmol/L) - 1 x 2.5mL bottle

7023-6302

Lactate Scout Check Solution High (8.9 - 11.1 mmol/L) - 1 x 2.5mL bottle

7023-6303

Lactate Scout Check Solution multi pack 1 x 2.5mL Low (1.8 - 2.2 mmol/L), 1 x 2.5mL Mid (4.5 - 5.6 mmol/L), 1 x 2.5mL High (8.9 - 11.1 mmol/L)



1
Remove test strip and place into analyser.



2
Position the tip of the Lactate Scout Test Strip on the sample site and collect blood by touching with test strip.



3
Result appears in 10 seconds.



Manufacturer

EKF-diagnostics GmbH
Ebendorfer Chaussee 3
39179 Barleben Germany
Tel: +49 (0) 39203 511 0
Email: sales@ekfdiagnostics.com
ekfdiagnostics.com

Distributed by